

Maintain protected
ureteral access.




Flexor[®]
URETERAL ACCESS SHEATH



www.cookmedical.com

The Flexor® Ureteral Access Sheath was one of the first sheaths on the market to have a reinforced coil construction to resist kinking and compression. The sheath provides ureteral dilation and a continuous working channel for the introduction of endoscopes and instruments during ureteral access procedures. When compared to several competitor sheaths, the Flexor was shown to be less likely to buckle while advancing up the ureter.¹

Use the Flexor to reduce the potential for trauma to the ureter during repeated instrument exchanges. The continuous working channel also protects delicate instruments and flexible endoscopes from damage, which can reduce the need for costly repairs.

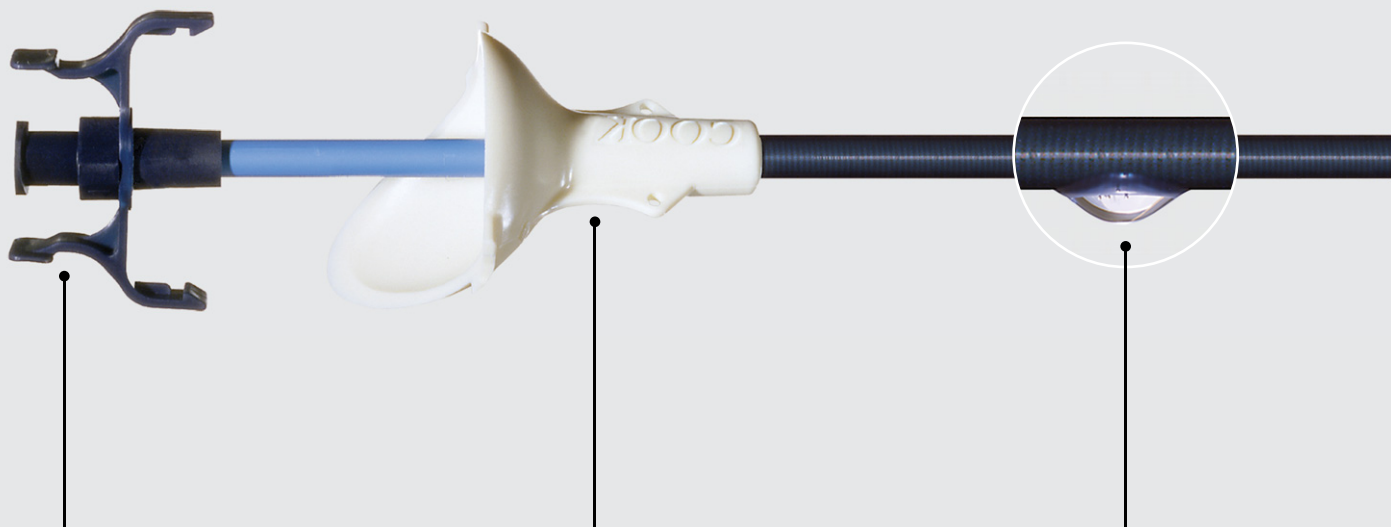

Dilator hub

The sheath's unique locking mechanism on the dilator hub secures the dilator to the sheath for simultaneous advancement of the sheath and dilator.

Funnel


The ergonomic funnel acts as a handle during insertion. The large trough facilitates instrument introduction.

Hydrophilic coating

The external surface of the sheath and the dilator tip are AQ® hydrophilic coated to create a low-friction surface to ease the insertion process.

Reliability

In a randomized study, the Flexor sheath proved to be reliable for gaining ureteral access for flexible ureteroscopy.²

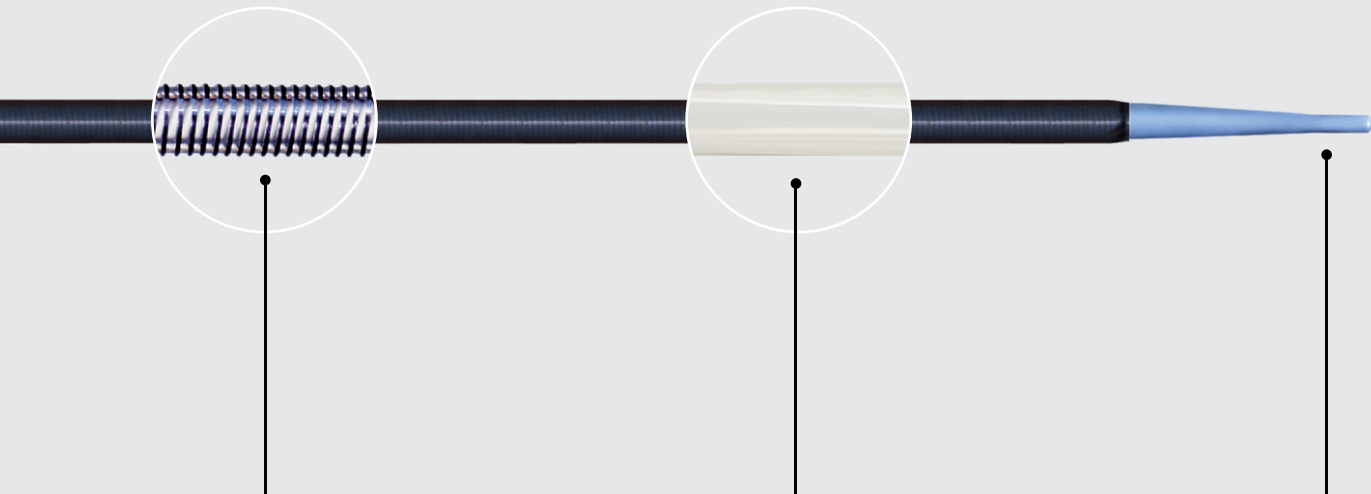

Core

The core consists of a patented coil construction to provide optimal flexibility and maximum resistance to kinking and compression.

Internal lumen

The internal lumen is PTFE lined to facilitate smooth device delivery and removal. Thin wall construction provides the largest possible internal lumen while minimizing the outer diameter.

Dilator

The dilator tapers smoothly from 6 Fr to provide gentle, one-step gradual dilation.

1. Pedro RN, Hendlin K, Durfee WK, et al. Physical characteristics of next-generation ureteral access sheaths: buckling and kinking. *Urology*; 70(3):440-442.

2. Monga M, Best S, Venkatesh R, et al. Prospective randomized comparison of 2 ureteral access sheaths during flexible retrograde ureteroscopy. *J Urol*; 172(2):572-573.

Flexor®

URETERAL ACCESS SHEATH

Order Number	Reference Part Number	ID Fr	Length cm
G32853	FUS-095013*	9.5	13
G19177	FUS-095020	9.5	20
G19176	FUS-095028	9.5	28
G19174	FUS-095035	9.5	35
G46167	FUS-095045	9.5	45
G19175	FUS-095055	9.5	55
G34876	FUS-107013*	10.7	13
G34877	FUS-107020	10.7	20
G34878	FUS-107028	10.7	28
G34879	FUS-107035	10.7	35
G34880	FUS-107045	10.7	45
G34881	FUS-107055	10.7	55
G32852	FUS-120013*	12.0	13
G19173	FUS-120020	12.0	20
G19172	FUS-120028	12.0	28
G19168	FUS-120035	12.0	35
G46168	FUS-120045	12.0	45
G19169	FUS-120055	12.0	55
G30391	FUS-140020	14.0	20
G30393	FUS-140028	14.0	28
G30389	FUS-140035	14.0	35
G46169	FUS-140045	14.0	45
G30390	FUS-140055	14.0	55

* For pediatric use in children 2 years of age and older.

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Customer Service - Urology

EMEA: EDI - www.cookmedical.com/edi.do
 Distributors: +353 61239240, ssc.distributors@cookmedical.com
 Austria: +43 179567121, oe.orders@cookmedical.com
 Belgium: +32 27001633, be.orders@cookmedical.com
 Denmark: +45 38487607, da.orders@cookmedical.com
 Finland: +358 972519996, fi.orders@cookmedical.com
 France: +33 171230269, fr.orders@cookmedical.com
 Germany: +49 6950072804, de.orders@cookmedical.com
 Hungary: +36 17779199, hu.orders@cookmedical.com
 Ireland: +353 61239252, ie.orders@cookmedical.com
 Italy: +39 0269682853, it.orders@cookmedical.com
 Netherlands: +31 202013367, nl.orders@cookmedical.com
 Norway: +47 23162968, no.orders@cookmedical.com
 Poland: +48 223060159, pl.orders@cookmedical.com
 Spain: +34 912702691, es.orders@cookmedical.com
 Sweden: +46 858769468, se.orders@cookmedical.com
 Switzerland - French: +41 448009609, fr.orders@cookmedical.com
 Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
 Switzerland - German: +41 448009609, de.orders@cookmedical.com
 United Kingdom: +44 2073654183, uk.orders@cookmedical.com
www.cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
 E-mail: orders@cookmedical.com

Australia:
 Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283
 E-mail: cau.custserv@cookmedical.com

